

HIV DRUG RESISTANCE

WHO MANUAL FOR HIV DRUG RESISTANCE TESTING USING DRIED BLOOD SPOT SPECIMENS

THIRD EDITION — OCTOBER 2020



WHO MANUAL FOR HIV DRUG RESISTANCE TESTING USING DRIED BLOOD SPOT SPECIMENS

THIRD EDITION — OCTOBER 2020

WHO manual for HIV drug resistance testing using dried blood spot specimens, third edition

ISBN 978-92-4-000942-4 (electronic version)

ISBN 978-92-4-000943-1 (print version)

© World Health Organization 2020

Some rights reserved. This work is available under the Creative Commons Attribution-NonCommercial-ShareAlike 3.0 IGO licence (CC BY-NC-SA 3.0 IGO; <https://creativecommons.org/licenses/by-nc-sa/3.0/igo>).

Under the terms of this licence, you may copy, redistribute and adapt the work for non-commercial purposes, provided the work is appropriately cited, as indicated below. In any use of this work, there should be no suggestion that WHO endorses any specific organization, products or services. The use of the WHO logo is not permitted. If you adapt the work, then you must license your work under the same or equivalent Creative Commons licence. If you create a translation of this work, you should add the following disclaimer along with the suggested citation: "This translation was not created by the World Health Organization (WHO). WHO is not responsible for the content or accuracy of this translation. The original English edition shall be the binding and authentic edition".

Any mediation relating to disputes arising under the licence shall be conducted in accordance with the mediation rules of the World Intellectual Property Organization (<http://www.wipo.int/amc/en/mediation/rules/>).

Suggested citation. WHO manual for HIV drug resistance testing using dried blood spot specimens, third edition. Geneva: World Health Organization; 2020. Licence: CC BY-NC-SA 3.0 IGO.

Cataloguing-in-Publication (CIP) data. CIP data are available at <http://apps.who.int/iris>.

Sales, rights and licensing. To purchase WHO publications, see <http://apps.who.int/bookorders>. To submit requests for commercial use and queries on rights and licensing, see <http://www.who.int/about/licensing>.

Third-party materials. If you wish to reuse material from this work that is attributed to a third party, such as tables, figures or images, it is your responsibility to determine whether permission is needed for that reuse and to obtain permission from the copyright holder. The risk of claims resulting from infringement of any third-party-owned component in the work rests solely with the user.

General disclaimers. The designations employed and the presentation of the material in this publication do not imply the expression of any opinion whatsoever on the part of WHO concerning the legal status of any country, territory, city or area or of its authorities, or concerning the delimitation of its frontiers or boundaries. Dotted and dashed lines on maps represent approximate border lines for which there may not yet be full agreement.

The mention of specific companies or of certain manufacturers' products does not imply that they are endorsed or recommended by WHO in preference to others of a similar nature that are not mentioned. Errors and omissions excepted, the names of proprietary products are distinguished by initial capital letters.

All reasonable precautions have been taken by WHO to verify the information contained in this publication. However, the published material is being distributed without warranty of any kind, either expressed or implied. The responsibility for the interpretation and use of the material lies with the reader. In no event shall WHO be liable for damages arising from its use.

CONTENTS

Acknowledgements	iv
Summary of changes	v
Introduction	1
Laboratory supplies	4
Preparing DBS from specimens collected by venepuncture	4
Preparing DBS from specimens collected by finger-prick	5
Drying	5
Important considerations for packaging DBS	6
Packaging	6
Handling and storage	6
Transport	7
Transport of DBS specimens stored at ambient temperature	8
Transport of frozen DBS specimens	8
Receipt of DBS at the processing laboratory	8
Using appropriate control reagents for HIV drug resistance testing from DBS	9
Validating HIV drug resistance testing from DBS	9
References	10
Annex 1. RNA extraction and genotyping protocols from the French National Research Institute for Sustainable Development	12
Annex 2. RNA extraction and genotyping protocols from the ILB Laboratory of the United States Centers for Disease Control and Prevention	19
Annex 3. RNA extraction and genotyping (Thermo Fisher Scientific kit) protocols from the ILB Laboratory of the United States Centers for Disease Control and Prevention	45

ACKNOWLEDGEMENTS

Neil Parkin (Data First Consulting, consultant, WHO Department of Global HIV, Hepatitis and STI Programmes) prepared this revision of the WHO manual for HIV drug resistance testing using dried blood spot specimens under the coordination of Silvia Bertagnolio (WHO Department of Global HIV, Hepatitis and STI Programmes). Many members of the WHO HIVResNet HIV Drug Resistance Laboratory Network and HIVResNet Laboratory Working Group reviewed the document, and their input is gratefully acknowledged.

SUMMARY OF CHANGES

Emphasis has been added on best practices for DBS preparation, storage and shipping, as well as selection of appropriate positive control. A new procedure for the ThermoFisher genotyping kit has been added (Annex 3).

预览已结束，完整报告链接和二维码如下：

https://www.yunbaogao.cn/report/index/report?reportId=5_24305

